

USER MANUAL



3-SECTIONED PATIENT EMERGENCY TROLLEY TABLE

For model
B1-130-1100

NOTE!

**The operator must read the user manual
carefully BEFORE the product is used,
to avoid unnecessary risks to the patient and operator.**

TABLE OF CONTENTS

Foreword

Warranty

Important information

1. Function Description

1.1 General description

2. Operation

2.1 How to be seated

2.2 Height adjustment

2.3 Adjusting the backrest

2.4 Adjusting the Trendelenburg position

2.5 Adjusting the leg part

2.6 The function of the castors

2.7 Side rails

2.8 IV stand

2.9 Storage basket

3. Maintenance & Cleaning

3.1 Routine check-ups

3.2 Cleaning

3.3 Lubrication scheme

3.4 Spare parts

3.5 Accessories

3.6 Solutions for lubrication, cleaning and disinfecting

4. Technical specification

4.1 Dimensions

4.2 Labels

5. EU-Declaration of Conformity

6. Contact details

FOREWORD

Welcome as the owner of a product manufactured by Sjöbloms Sjukvårdsutrustning AB. We hope that you will be pleased and satisfied with our quality product.

If a problem should arise with this or any of our products, you are always welcome to contact us, and our service department will assist you with the expertise needed to solve the problem.

The user manual contains all the information needed to use, maintain and service the product. The document is continuously updated as the products are modified. Therefore we reserve against possible changes in the user manual.

WARRANTY

Sjöbloms provides a two-year warranty for material, design and manufacturing defects, provided the products are not subjected to tampering or excessive wear. For the guarantee to apply, the product must have been used in accordance with the user manual, and the maintenance performed as specified.

IMPORTANT INFORMATION

This product is intended for professional medical use. The user manual should be read carefully before using the product, to avoid unnecessary risks for the patient and operator.

The user manual contains warnings where there is a risk of personal injury, which should be noted. Improper handling can be fatal.

A warning triangle is shown where particularly important information can be found.



Warning triangle

1. FUNCTIONAL DESCRIPTION

This product is meant for professional use in health care, for examination of the patient.

It is designed to be a functional and user-friendly tool for the operator and patient.



Note that the product must only be used by competent health professionals who read the user manual, for correct handling.

General description


3-sectioned patient emergency trolley, hydraulically adjustable, chromium-plated all-welded steel tube chassis. Backrest, legrest and Trendelenburg adjusted by gas springs.

Removable pad, available in different designs.

150 mm swivel central locking castors with a directional locking function. Various accessories such as neck pillow, IV stand etc. are also available.

2. OPERATION

2.1 How to be seated

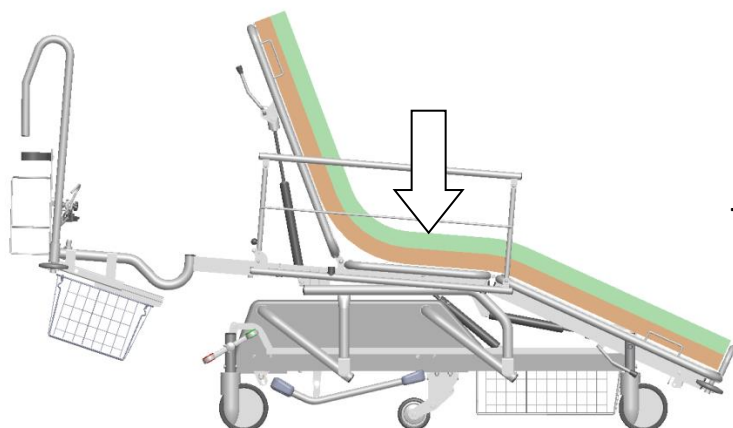

For maximal security the patient is recommended to be seated in the middle of the trolley (see arrow).



The castors are to be locked before being seated.

Fold down one of the siderails so that the patient may be seated properly. The patient is to go on and off the table from the side.



NOTE!

Be aware so that clothes don't get stuck on the castor attachment tube (A).

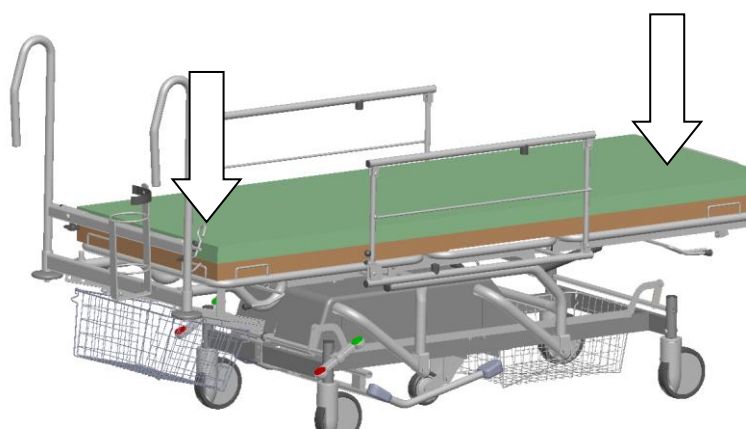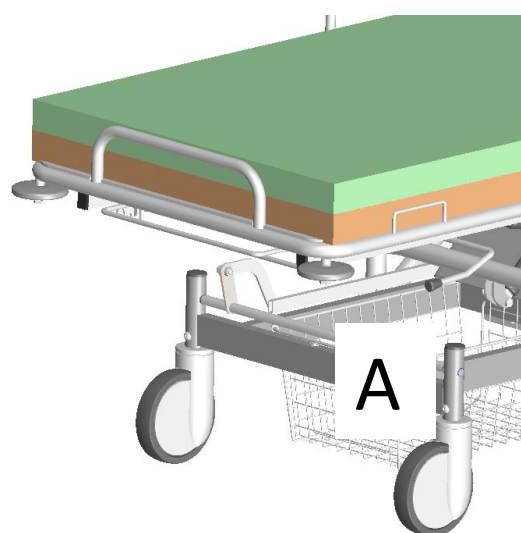

NOTE!

The patient must not sit at the foot or head end (arrows left). Risk of tipping.




NOTE!

Ensure that no unauthorized persons are in the couch's working range, as a pinch hazard exists when raising and lowering the product.

2.2 Height adjustment


Upwards

Pump with your foot on the longer part of the pedal until desired height is reached.

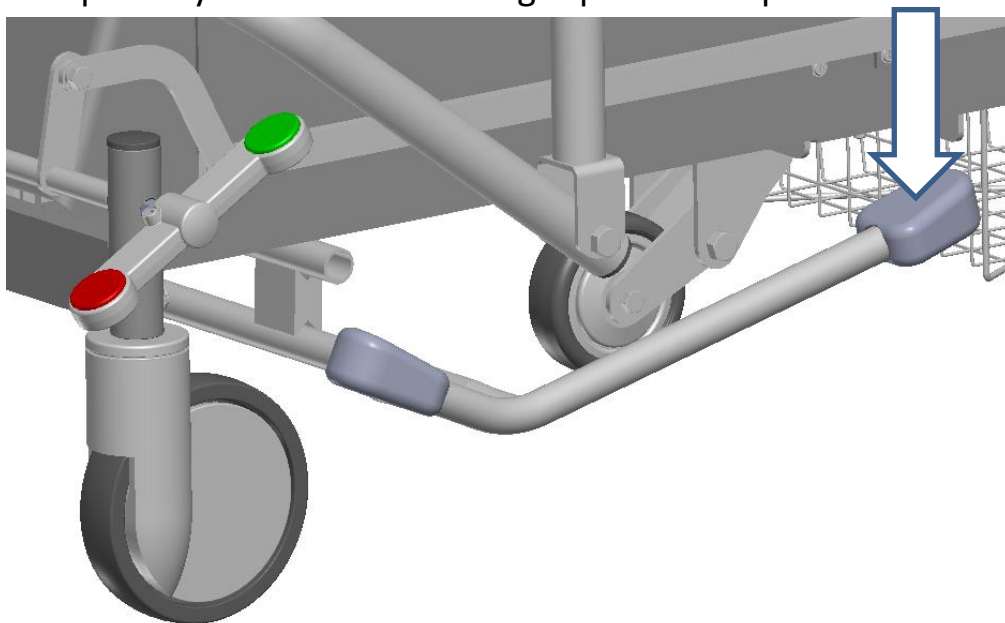

Downwards

Press the shorter part of the pedal and hold until desired height is reached.

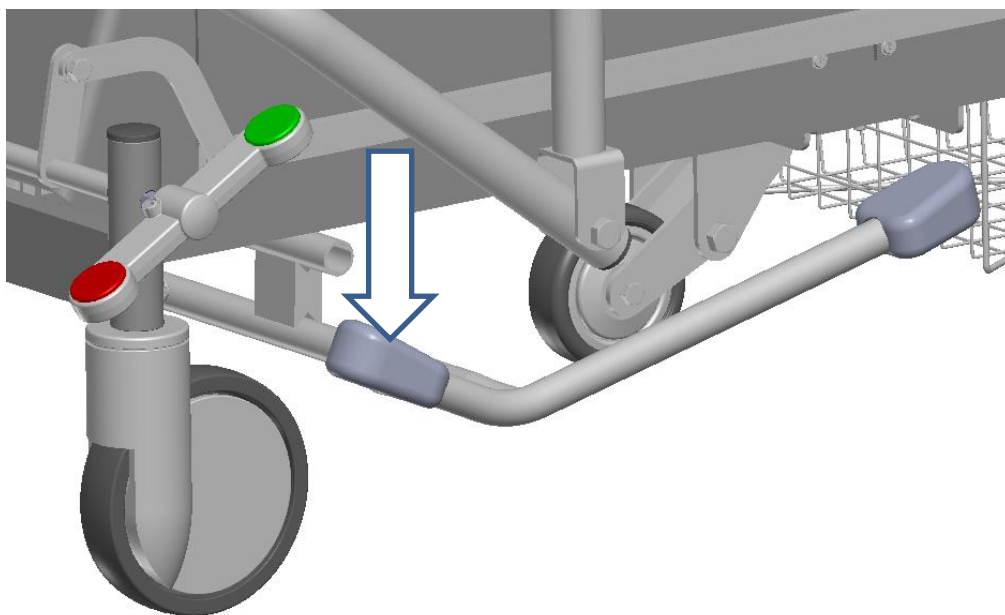


Varning!

Se till att inga obehöriga befinner sig inom patientvagnens arbetsområde, då klämrisk föreligger vid höjning och sänkning.

2.3 Adjusting the backrest

Raising the backrest

Take hold of at least one of the handles placed on the backrest.

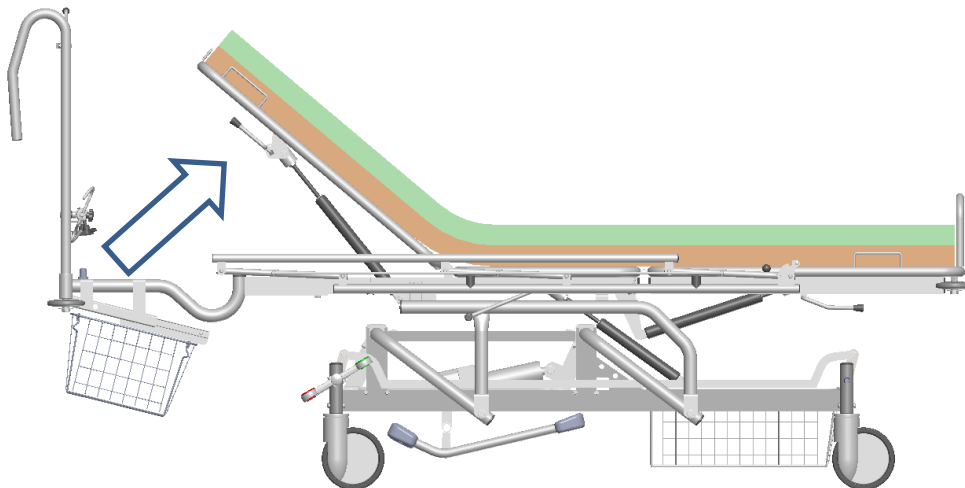

Press the handle inwards while holding. The gas spring helps adjust the backrest upwards variably.

Lowering the backrest

Press the handle inwards while holding. With your other hand, help press the backrest downwards.




NOTE!

Ensure that no unauthorized persons are in the couch's working range, as a pinch hazard exists when raising and lowering the product.


2.4 Adjusting the Trendelenburg position

Upwards

Stand at the foot-end side of the trolley. Take hold of the handle (see lower image).

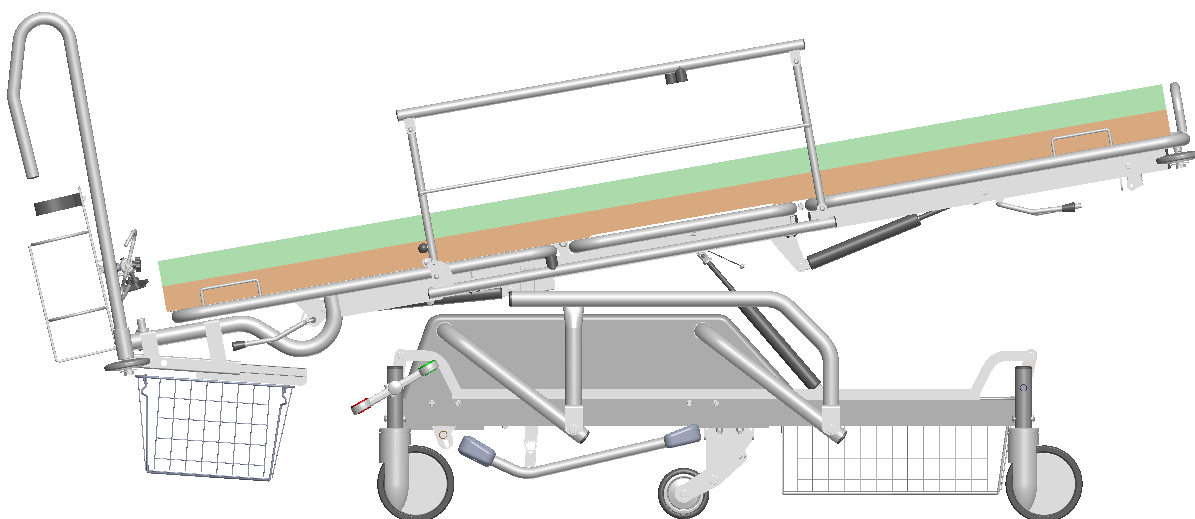

Press the handle upwards. The gas spring helps adjusting the trolley to Trendelenburg or anti-Trendelenburg position.

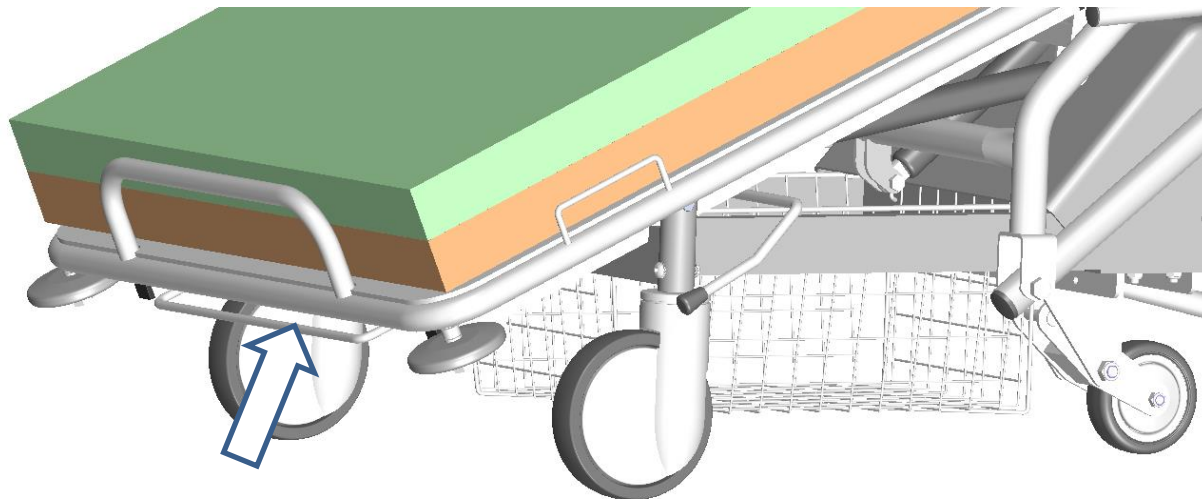


Downwards

Press the handle upwards while holding and help press the top and pad downwards.



Ensure that no unauthorized persons are in the couch's working range, as a pinch hazard exists when raising and lowering the product.

2.5 Adjusting the legrest

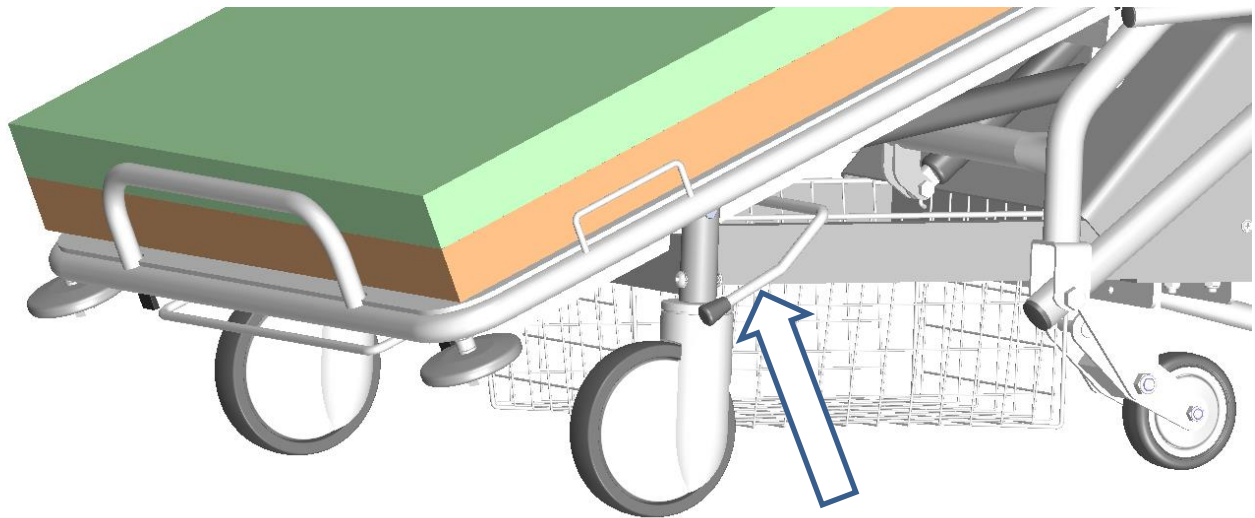

Upwards

Press the handles placed on the side. The gas spring helps adjust the legrest upwards. Let go when desired angle is reached.

Downwards

Press the handle upwards while holding and help press the top and pad downwards.



NOTE!


Ensure that no unauthorized persons are in the couch's working range, as a pinch hazard exists when raising and lowering the product.

2.7 The function of the castors


The product is equipped with central locking castors.

150 mm swivel central locking castors with a directional locking function, to help with straight and longer transports. As an option, the trolley is available with a fifth steering wheel, which helps manouvering the trolley with larger precision on a small amount of space.

Braked castors

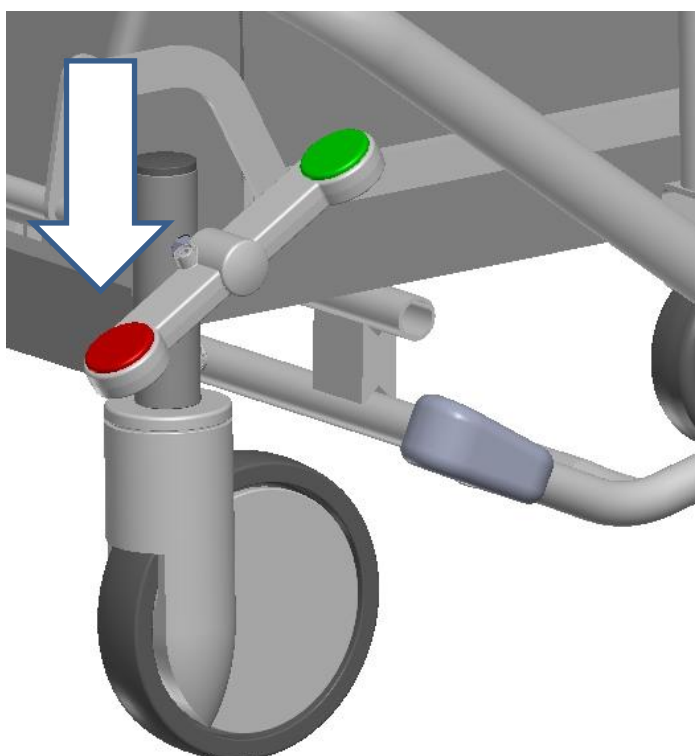



Directional locking function or steering wheel activation

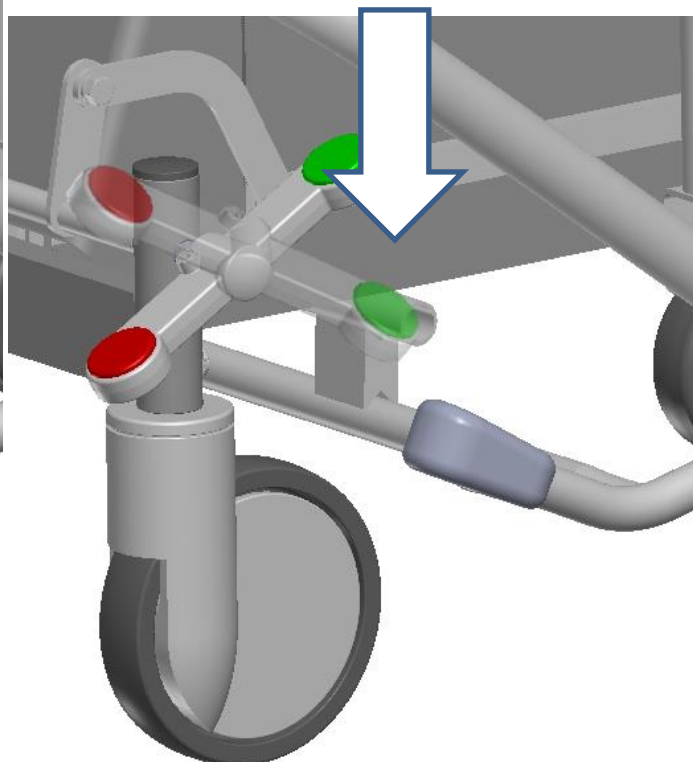


The fifth steering wheel



2.7 Side rails

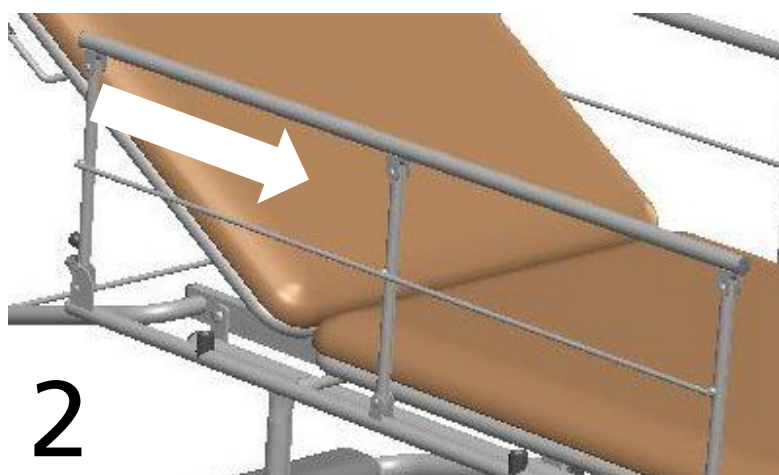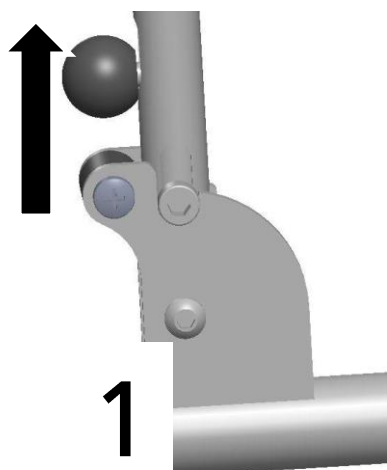


1. Lift the black ball.
2. Fold the siderails using the other hand in the direction of the arrow.
3. Done!
4. When raising the siderails, make sure its mechanic clicks into position indicating it is fully raised.

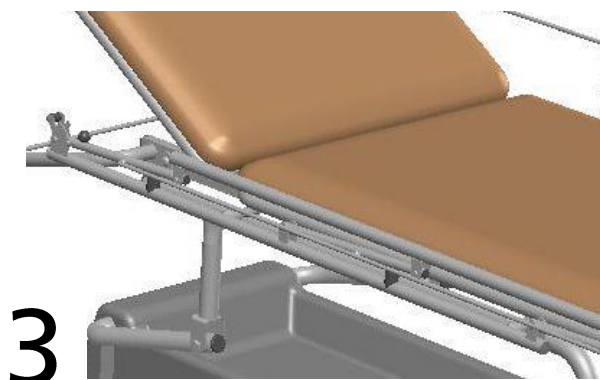

NOTE!

Pinch hazard exists when raising and lowering the side rails.

2.9 IV stand

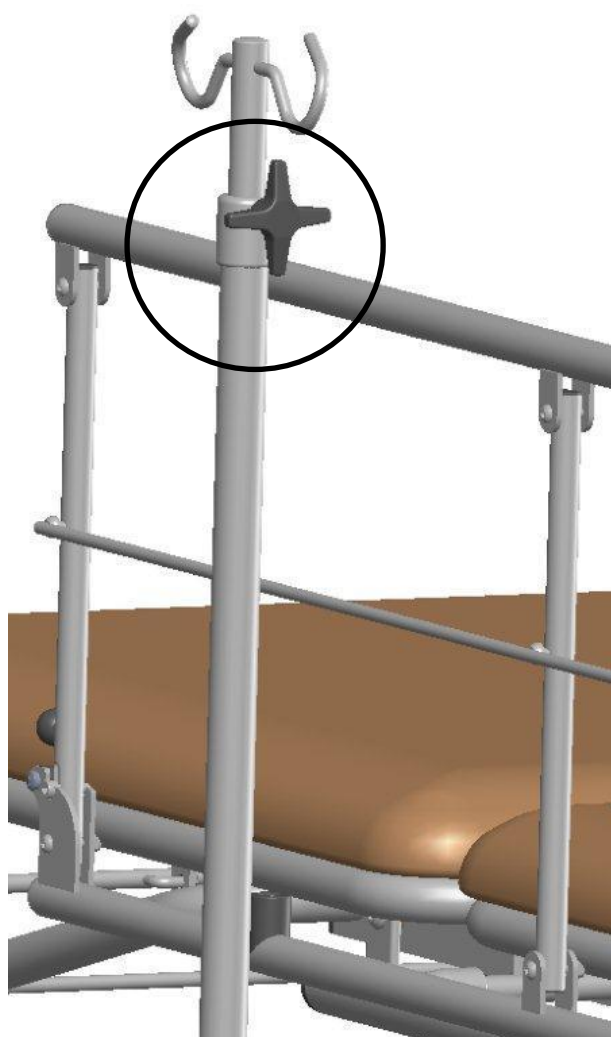

The IV stand may be adjusted in height.

Lock/open the knob to adjust the height.

2.9 Storage basket



The storage basket can be pulled out.

3. MAINTENANCE & CLEANING

3.1 Routine check-ups

Routine check-ups should be made to ensure proper functionality.



Important!

Failure to perform the following routine can cause the patient and/or operator to be exposed to unnecessary risks and the couch may have a reduced life expectancy. Please check all functions and parts on a regular basis.

Check the following:

- Any unusual sounds.
- Gas spring functions.
- Castors and castor locking mechanism.
- On electric tables:
 - Damages to electric cables.
 - The motor functions.
- That joints and fittings are lubricated according to the lubrication chart.

3.2 Cleaning

Cleaning of imitation leather

The examination table is very easy to clean. During normal cleaning, warm water and a mild soap solution should be used, or our own “Cleaning solution for artificial leather”, **art no. R-1**. Stronger cleaning agents should be avoided whenever possible. If it nonetheless becomes necessary to use stronger agents, it is important to wipe with a damp cloth.


Disinfection

The product could be disinfected by general hospital-used solutions.

Note that a too strong alcohol solution can have a drying effect, which means that the imitation leather's properties deteriorate and cracks can form.

3.3 Lubrication


Lubrication according to the scheme should be performed every 6 months or as needed.

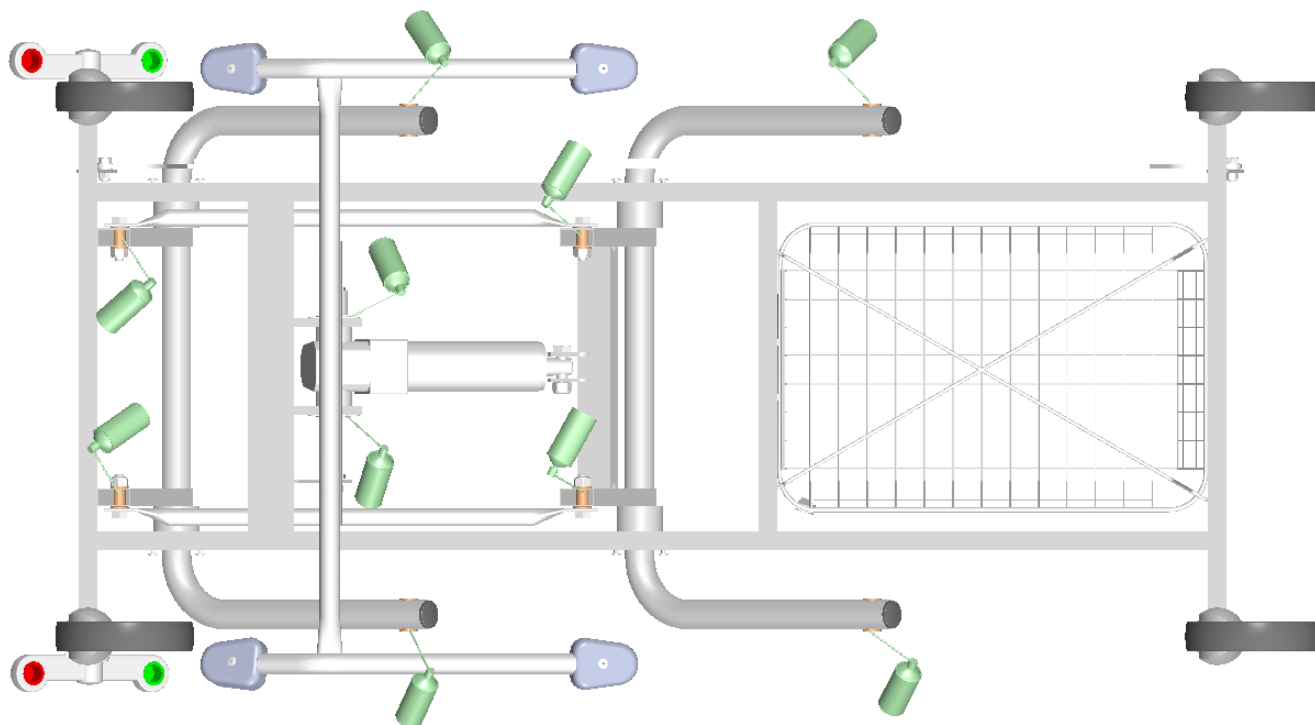

1. First lock the castors.
2. Enlist the help of another person and tip the couch on the side, in order to lubricate the moving joints.

The spray can in the pictures above indicates where to lubricate.

3.4 Spare parts

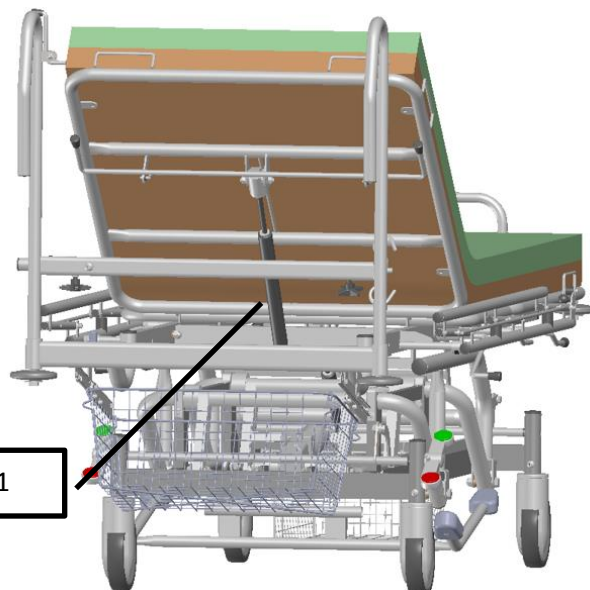


No	Art no	Item
1	G0002	Gas spring for backrest and legrest
2	G0003	Gas spring for Trendelenburg
3	00027848	Castor
4	00032427	Castor, directional locking
5	HY001	Hydraulic pump
6	10032100000RP	Fifth steering wheel



3



4

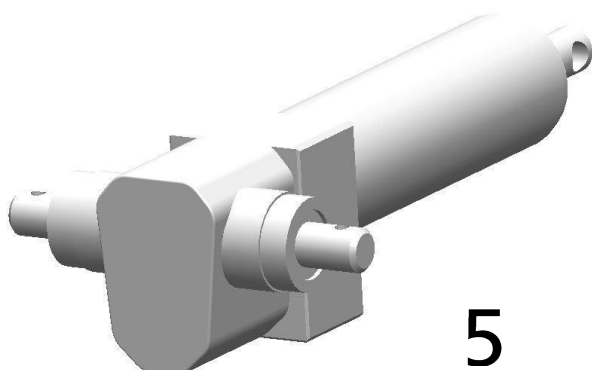

5



6

3.6 Accessories



Side rails



IV stand



Neck cushion

3.7 Solutions for lubrication, cleaning and disinfecting

R-1, Cleaning solution for artificial leather

5557001, Lubrication

C068, Disinfection solution, DAX Plus



Cleaning solution



Lubrication



Disinfection

4. TECHNICAL SPECIFICATIONS

4.1 Dimensions

Total length: 225 cm

Patient length: 200 cm

Patient width: 70 cm

Width including the bumper wheels: 76 cm

Width including siderails: ca 80 cm

Height: 52-90 cm excluding pad

Length backrest: 75 cm

Backrest angle: -14° , $+4^{\circ}$


Trendelenburg angle: -14°

Weight: 94 kg

Max patient weight: 200 kg

SWL: 220 kg

4.5 Labels on the product

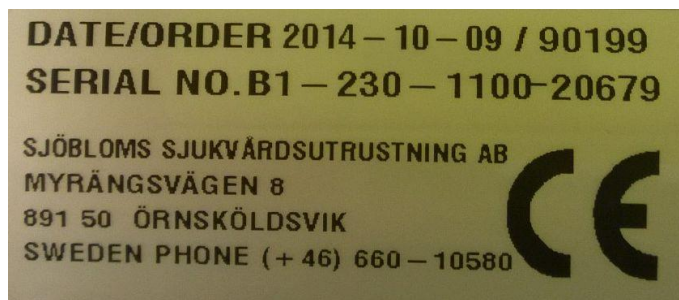


The CE label contains the date, order number, serial number and our contact information.



Max patient weight 200 kg.



Displays the adjustment of the backrest. There is a label on each side of the table.

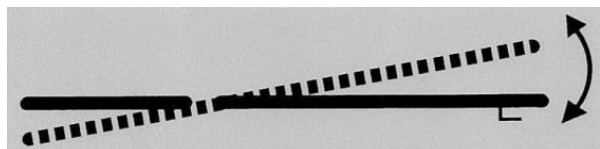

Displays the adjustment of the Trendelenburg position. There is a label on each side of the table.

5. EU-DECLARATION OF CONFORMITY

EU-DECLARATION OF CONFORMITY

according to MDD 2007/42/EC including Appendix
and according to CE-marking Directive 93/68/EEC

Type of equipment: Patient emergency trolley, hydraulic

Model: B1-130-1100 Product registration number: 74550

Manufacturer

Sjöbloms Medical Equipment AB
Myrängsvägen 8
S-891 50 ÖRNSKÖLDSVIK, SWEDEN

Sjöbloms Sjukvårdsutrustning AB ensures that the above products are in accordance with the EU Council MDD 93/42/EEC, as well as the Swedish legislation in force (1993:584) on medical devices and the regulations issued by Medical Products Agency (LVFS 2003:11).

Örnsköldsvik 2015-08-21



Karl-Gunnar Sjöblom
Production Manager

6. CONTACT INFORMATION

SJÖBLOMS

Myrängsvägen 8

891 50 Örnsköldsvik

Sweden

Tel: +46 660 842 51

Fax: +46 660 180 00

E-mail: info@sjobloms.com

Web page: www.sjobloms.com